ENGLISH

INTENDED USE/INDICATIONS:

Clavicle fractures, acromioclavicular separations, osteo-clavicular joint pain, and postural problems.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- Follow application and care process described in instructions.
- For single patient use only.

INSTRUCTIONS FOR USE:

1. Unfasten hook and loop closure.
2. Position Clavicle Brace such that the center of the brace is centered on the patient's back. The D-rings should rest furthest from the patient's neck.
3. Position straps around the top of the patient's shoulders and wrap under the arm pits. Thread the side straps through the D-rings.
4. Tighten the straps, to pull the shoulders back, and fasten the hook tabs on the ends of the straps to the brace.
5. The D-ring attachment site can be repositioned on the center strap for additional support and comfort.

USE AND CARE:

Fasten hook and loop straps prior to washing. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Nylon, Polypropylene, Polyurethane.

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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