

# Neoprene

## Hinged Knee Suport

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THIS DEVICE.

### INDICATIONS

The Neoprene hinged knee brace provides stability and support for chronic knee pains and instability. Effective in treating mild injuries of the collateral ligaments, menisci, and patellar retinaculum.

### CAUTION

The application instructions listed above are intended for the use of qualified professionals only. It is not recommended for patients to attempt to apply or adjust this device unless otherwise instructed to do so by an orthopedic professional. Any questions regarding further adjustment or frequency/ duration of wear should be directed to your orthopedic professional, as they know the individual condition for which this device was prescribed.

### APPLICATION

1. Remove the knee brace from its packaging and gently slip it up the leg and over the knee, making sure the sizing tag on the inside of the brace is at the top (Fig. A).
2. Position the knee brace so that the silicone padding fits comfortably over the knee. Beginning with the top closure strap, feed each strap through the d-ring provided and secure, making sure the brace fits snug (Fig. B).
3. Fasten the bottom closure strap, making sure that the silicone padding is still aligned with the knee. Adjust and tighten as needed (Fig. C).



### CLEANING INSTRUCTIONS

Remove hinges from brace and hand wash in cold water with mild detergent. Allow all parts to air dry before re-assembly. Do not dry clean, iron, or bleach any part of this brace, as it could cause damage and void the manufacturer's warranty.

NOTICE: While every effort has been made to obtain maximum strength, function, and durability, there is no guarantee that injury will be prevented through the use of this device.

Comfortland Medical, Inc.  
709 AO Smith Rd.  
Mebane, NC 27302  
T 866.277.3135 | F 866.277.4412  
[www.comfortlandmed.com](http://www.comfortlandmed.com)