
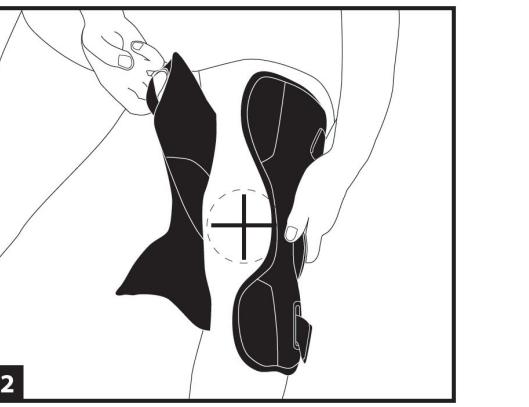
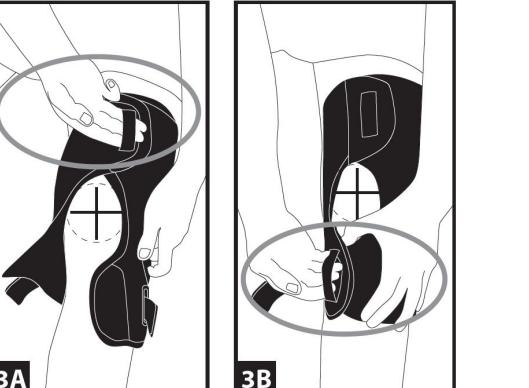
1


2

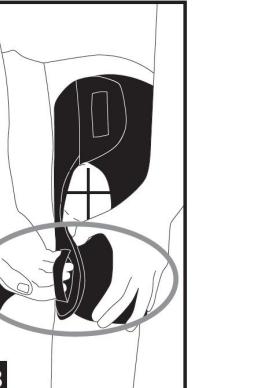
3A


3B

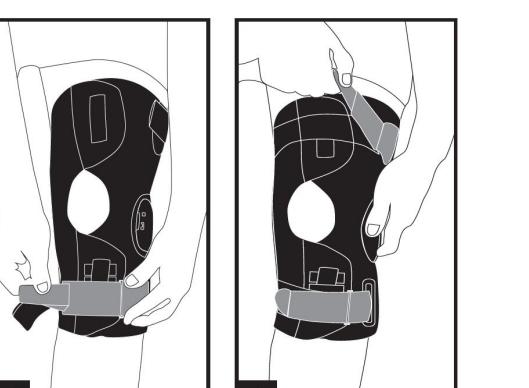4A

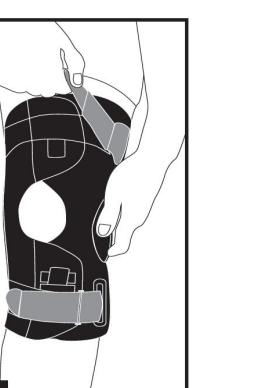4B

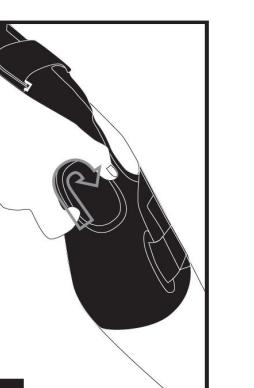5

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS: The DonJoy Clima-Flex OA is designed to provide support to the knee. It may be suitable for those suffering from moderate uni-compartmental (one compartment) knee osteoarthritis. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained within this product.

WARNINGS AND PRECAUTIONS:

- This device is to be prescribed and fitted by a qualified healthcare professional.
- The frequency, duration of use and directions for use should be determined by your prescribing healthcare professional.
- If discomfort, pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use the device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- Unfasten straps and flaps so the brace is completely open.
- Align the center of hinge with the center of the knee.

NOTE: The hinge bar should be located on the unaffected side of the knee.

- Wrap the thigh flap first then the calf flap to secure brace. Adjust for a snug fit.
- Thread the straps through the D-Rings and tighten.

Hinge Adjustment:

- Rotate both knobs on hinge clockwise to increase off-load. Ensure the numbers are increasing as you turn the knobs. Adjust as needed to help relieve OA symptoms.

NOTE: Ensure no more than a 2-digit difference.

CLEANING INSTRUCTIONS:

Hand wash with soap in 30°C water.

Do not bleach. Do not iron.

MATERIAL CONTENTS: Polyester, Aluminum, Nylon, PVC, Carbon, Polycarbonate, Suede, Silicone

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

Rx ONLY.**FOR SINGLE PATIENT USE ONLY.**

NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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