Patient Type	Neck Height	Circumference	Circumference Adjustment
Stout	2 (2.25" / 5.7 cm)	Large (16"-19" / 41 - 48 cm)	No tear off
Average Male	3 (3.25" / 8.3 cm)	Medium (13"-16" / 33 - 41 cm)	Tear off 1 section on each side
Thin Female	4 (4.25" / 10.8 cm)	Small (10"-13" / 25 - 33 cm)	Tear off 2 sections on each side

Spine Application

Ensures that correct spinal protocol is followed:

- Slide the back piece behind the patient's neck and center it, noting arrow on panel pointing up (Fig. 5).
- Note: Long hair should be placed outside of the back piece.
- Apply the front piece with the chin secured in the recess and center it to secure neutral alignment. The front piece overlaps the back piece to ensure effective immobilization and comfort (Fig. 6). The front arrow should point upward.
- Apply the Hook and Loop strap on the equal length on both sides (Fig. 7). Excess may be trimmed.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIAIBILITY

Ossur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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Device Removal

Ensure that correct spinal protocol is followed.

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GENERAL SAFETY INSTRUCTIONS

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

GENERAL CAVITY INJURIES

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

General Contraindications

• Patients with a compromised airway or known spinal deformities such as ankylosing spondylitis.

• Patients with penetrating trauma injuries.

Warnings and Cautions:

Warning: If an unstable fracture is suspected or unknown, with or without a suspected airway, ensure additional precautions are implemented to immobilize the spine.

Warning: Use of a cervical collar may increase intracranial pressure (ICP) through jugular venous compression.

Warning: Use of a cervical collar may increase complexity of airway management.

Warning: Cervical spine immobilization, including use of a cervical collar, has been associated with:

- Impaired respiratory effort and forced expiratory volume.
- Pneumonia.
- Aspiration.
- Development of existing cervical spine injury.
- Severe neurological deterioration in patients with ankylosing spondylitis.
- Triggering of non-compliance or agitation.
- Delay to definitive treatment.
- Impaired physical examination/secondary survey.

Caution: Cervical spine immobilization, including use of a cervical collar, has been associated with:

- Increased pain and discomfort, which may lead to increased spinal movement.
- Impaired swallowing.

Caution: Perfumes and harsh cleaning agents should not be used on or under the collar, as they may compromise the integrity of the materials.

Caution: Use of a cervical collar for cervical spine immobilization is not recommended in situations where patients are aware, alert, not intoxicated, without neck tenderness or pain and without abnormal sensory or motor findings on examination.

Caution: Redraw of the collar, pads and neck skin should be done with caution of the skin for any signs of irritation is required to reduce the risk of skin ulceration. Bedridden patients are at an increased risk for skin ulceration.

ENGLISH

SYMBOLS

MD

Medical Device

MR

Magnetic Resonance (MR) safe

INTENDED USE

The device is intended to provide gross immobilization to the cervical spine. The device must be fitted and adjusted by a healthcare professional.

X-ray and CT fluent.

Indications for use

Conditions requiring gross immobilization of the cervical spine. This may include:

- C-Spine precaution for trauma patients.
- Immobilization for pre and post surgery.

Contraindications

• Patients with a compromised airway or known spinal deformities such as ankylosing spondylitis.

• Patients with penetrating trauma injuries.

Warnings and Cautions:

Warning: If an unstable fracture is suspected or unknown, with or without a suspected airway, ensure additional precautions are implemented to immobilize the spine.

Warning: Use of a cervical collar may increase intracranial pressure (ICP) through jugular venous compression.

Warning: Use of a cervical collar may increase complexity of airway management.

Warning: Cervical spine immobilization, including use of a cervical collar, has been associated with:

- Impaired respiratory effort and forced expiratory volume.
- Pneumonia.
- Aspiration.
- Development of existing cervical spine injury.
- Severe neurological deterioration in patients with ankylosing spondylitis.
- Triggering of non-compliance or agitation.
- Delay to definitive treatment.
- Impaired physical examination/secondary survey.

Caution: Cervical spine immobilization, including use of a cervical collar, has been associated with:

- Increased pain and discomfort, which may lead to increased spinal movement.
- Impaired swallowing.

Caution: Perfumes and harsh cleaning agents should not be used on or under the collar, as they may compromise the integrity of the materials.

Caution: Use of a cervical collar for cervical spine immobilization is not recommended in situations where patients are aware, alert, not intoxicated, without neck tenderness or pain and without abnormal sensory or motor findings on examination.

Caution: Redraw of the collar, pads and neck skin should be done with caution of the skin for any signs of irritation is required to reduce the risk of skin ulceration. Bedridden patients are at an increased risk for skin ulceration.

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